

SNORE^RPRO
X

Oral Profile Instructions

The Solution for Snoring.

Before you begin...

Be prepared to work quickly to take a good oral profile. Review these instructions and make sure you have everything you need before you get started.



Brush your Teeth.

Lunchtime leftovers can ruin your oral profile!



Wash your hands.

Always clean your hands before touching your mouth.



Get a timer ready.


Oral profile polymer is time-sensitive.



Grab a towel.

This could get a little messy...

Here's how your oral profile should look:

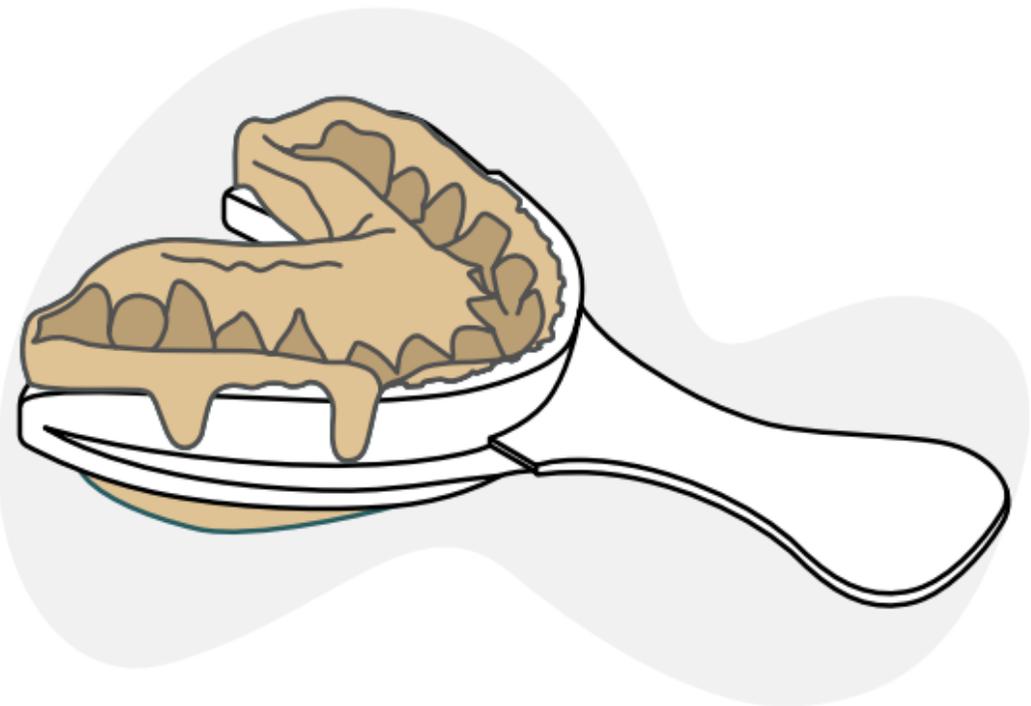

Watch video instructions at:


▶ www.snorerx.com/instructions



Mix ⌚ ~60 SEC.

Remove the polymer from both containers and knead them together to make a light blue polymer.


- ⓘ This polymer will harden quickly after you mix it, so do not knead for any longer than 60 seconds.

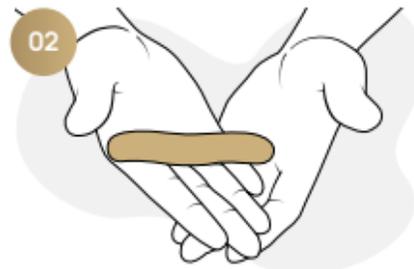

Roll ⌚ 10-20 SEC.

Quickly roll the polymer into a 4-inch-long cylinder.


- ⓘ Do not spend too much time on this step! This should take no more than 20 seconds.

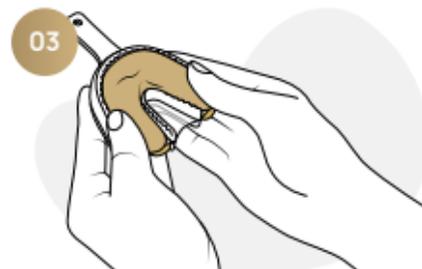

Place ⌚ 20-30 SEC.

Divide the polymer in half, and gently press one piece into one side of the tray. Next, press the other piece into the other side of the tray.

- ⓘ Make sure the polymer completely fills the tray, but do not press too hard.

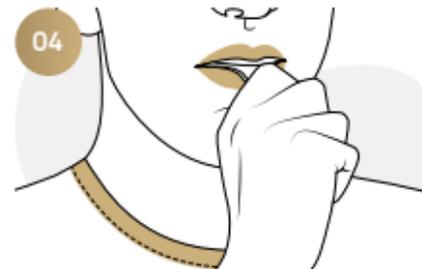

Bite ⌚ 4 MIN.

Place the tray in your mouth and use your fingers to center the tray evenly over all of your teeth. Bite firmly into the polymer and press your upper and lower teeth together. After 4 minutes, take out the tray, and rinse your mouth.



Write

Write your order number and contact information on the registration form provided. Place this form in the envelope with your oral profile. If you have not already purchased SnoreRx Pro, visit www.snorerx.com/register



Send

Place the profile tray and registration card into the prepaid envelope included with your order. Seal the envelope, and drop it in the mail.

- ⓘ Do not separate your oral profile from the tray!

FAQs

How do I send in my oral profile?

We included a prepaid and self-addressed envelope with your order. Simply place your oral profile and completed registration form in this envelope and place it in any United States Postal Service mailbox. We'll take care of the rest!

What if I mess up while taking my oral profile?

Mistakes happen! We understand. Just email us at support@snorerx.com ASAP, and we'll send you a new oral profile kit. Please be aware that there may be a fee for the replacement materials.

How do I know if my oral profile is good?

Our dental professionals will check your profile, and then we will send you an email to let you know whether your oral profile was approved. If your profile didn't come out right, we will contact you as soon as possible.

Have more questions?

Get in touch at:
[**support@snorerx.com**](mailto:support@snorerx.com)

Part No. SNPR26 Rev 1
Effective Date: 3/1/21

www.snorerx.com